

# Thera-Band® Resistance Bands and Tubing Exercises

### Shoulder

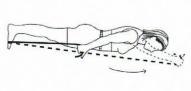


### Shoulder Flexion\*

- Stand on elastic.
- Begin with arm at side, elbow straight, thumb up.
- Grasp elastic.
- Raise arm in front over head, keeping elbow straight.
- Slowly return to starting position.

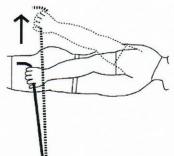

### **Shoulder Extension**


- Secure elastic at waist level as shown.
- Grasp elastic and pull arm backwards keeping elbow
- Slowly return to start position

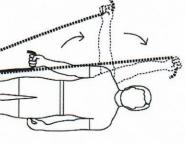

### Shoulder Abduction\*

- Stand on elastic.
- Begin with arm at side, elbow straight, holding elastic, palm forward
- Raise arm upward, out to side and over head.
- Slowly return to starting position.




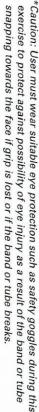


# **≡Thera-Band≡** Instruction Manual

### **Shoulder Adduction**

- Attach elastic to secure object at waist level
- Grasp elastic in hand.
- Pull arm inward, keeping elbow straight.
- Slowly return to start position and repeat.

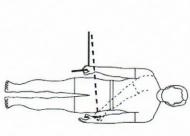

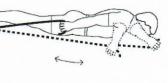


## Scaption (Abduction in Scapular Plane)

- Stand on elastic.
- Begin with arm at side, elbow straight, holding elastic, thumb up
- Raise arm in a position halfway between the front and side, over head.
- Slowly return to starting position.



### Shoulder Internal Rotation


- Secure elastic at waist level.
- Sit or stand with involved side to elastic, elbow at 90 degrees, arm at side.
- Grasp elastic and pull hand inward, across body, as
- Slowly return to start position and repeat.

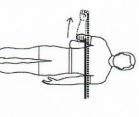

### **Shoulder External Rotation**

- Attach elastic to secure object at waist level.
- Place pillow between elbow and body.
- Grasp elastic in hand, elbow bent to 90 degrees
- Rotate arm outward and return.
- Slowly return to start position and repeat.

# **≡**Thera-Band**≡** Instruction Manual

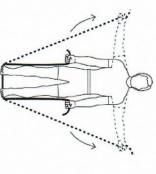


### Lateral Raise

- Stand on elastic.
- Hold elastic in both hands
- Begin with arms at sides.
- Position palms forward.
- Keep elbows straight and lift arms to shoulder level.
- Slowly lower and repeat.

#### Front Raise\*

- Secure elastic at waist level as shown.
- Grasp elastic and pull arm backwards keeping elbow straight.
- Slowly return to start position.

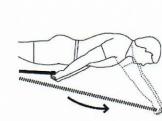




### Overhead Press\*

- Stand on elastic.
- Grasp elastic in hands, arms at side, elbows bent, as shown.
- Push arms up and overhead.
- Slowly return to start position and repeat.



### Shoulder Diagonal D1 Flexion\*

- Secure elastic to secure object at floor level.
- Sit or stand, arm at side.
- Grasp elastic in hand, palm forward.
- Lift arm upward and across body to opposite shoulder, bending elbow, ending with palm inward.
- Slowly return to start position and repeat.

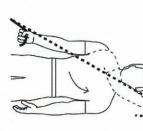

End with hand at opposite hip, palm inward.
Slowly return to start position and repeat.


Attach elastic to secure object above head level
Begin with arm up and out from side as shown.
Grasp elastic, palm forward and pull down


# \*Caution: User must wear suitable eye protection such as safety goggles during this exercise to protect against possibility of eye injury as a result of the band or tube snapping towards the face if grip is lost or if the band or tube breaks.

# **≡**Thera-Band**≡** Instruction Manual

### Shoulder Diagonal D2 Flexion\*


- Attach elastic to secure object at floor level.
- Begin with arm crossed over trunk, holding elastic, palm inward, at opposite hip.
- Raise arm up and diagonally across, ending with palm facing forward.
- Slowly return to start position and repeat

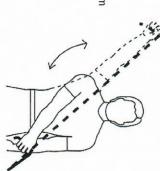

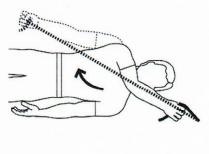

## Shoulder Diagonal D1 Extension\*

- Attach elastic to secure object as shown.
- Grasp elastic with hand at opposite shoulder, palm inward.
- Pull arm down and across body, ending in palm outward.
- Slowly return to start position and repeat.

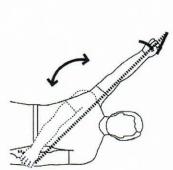

Shoulder Diagonal D2 Extension\*

<sup>\*</sup>Caution: User must wear suitable eye protection such as safety goggles during this exercise to protect against possibility of eye injury as a result of the band or tube snapping towards the face if grip is lost or if the band or tube breaks.